

# FIRST + CRUSH™

AUTOMATICALLY TURNS PILLS TO POWDER™



**TWO CUPS ARE REQUIRED TO REDUCE CONTAMINATION;  
PLEASE CHARGE DAILY FOR MINIMUM OF TWO HOURS**



1. Press **ON** button to turn on unit  
(GREEN LIGHT should be ON)



2. Place pills into a cup and  
second cup on top.



3. Push **OPEN** button and lift lid,



4. Place in cups into cup holder and  
firmly close lid until latches,



5. Push either **STANDARD** or **EXTRA GRIND**  
**GRIND** button.



6. Remove both cups and tap TOP  
cup to remove any residue.

# FIRST CRUSH <sup>TM</sup>

AUTOMATICALLY TURNS PILLS TO POWDER <sup>TM</sup>



**WHEN GRINDING IS FINISHED, ADD APPLESAUCE OR PUDDING DIRECTLY INTO BOTTOM CUP AND ADMINISTER**

## IMPORTANT NOTES:

**TURN OFF:** After your medicine pass is complete, hold down the **ON** button for a few seconds to turn off. Only use 6-8 small to medium sized pills at a time. Overloading the cup will lead to spills. For large pills use a maximum of 3-4 pills and **EXTRA GRIND** setting to ensure best results.

If pill is not crushed completely; use **EXTRA GRIND** setting and/or shake cup gently to move powder to outer edge of cup and re-crush.  
If motor stops before end of cycle, simply press the **STANDARD** button again to cycle.

## RECHARGING:

Plug power cord into back of unit.

- When unit is being charged - solid **RED** light is on
- When unit is fully charged - **RED** light turns off (no light on)

**RECOMMENDED: UNIT SHOULD BE CHARGED EVERY NIGHT FOR TWO HOURS TO ENSURE PROPER USE.**

## LIGHT INDICATORS:

- “ON” - solid green light
- “OFF” - no lights on
- Needs charging - blinking red light (slow blinking)
- Unit is about to stop running - red light blinks rapidly (noticeably faster than when charge battery light first comes on)
- Stops running - red light blinks most rapidly (when a cycle button is pushed, nothing happens)
- Battery charging - solid red light
- Battery fully charged - red light turns off (no light on)

## CLEANING:

Please unplug the power cord from the back of unit before cleaning to avoid potential injury.

Use a damp or lightly moist paper towel, baby wipe or equivalent to clean unit at the end of each medpass.

If powder spills out of the cup due to overloading:

1. Please remove bottom CUP HOLDER.
2. Pour any powder back into your medicine cup.
3. Clean the CUP HOLDER with soap and water.
4. Dry and place the CUP HOLDER back in the unit.
5. Grooves on the bottom of the CUP HOLDER must fit into the slots in the unit to ensure proper alignment.